# Zyto Light ® SPEC EWSR1 Dual Color Break Apart Probe



# **Background**

The ZytoLight® SPEC EWSR1 Dual Color Break Apart Probe is designed to detect translocations involving the chromosomal region 22q12.2 harboring the EWSR1 (EWS RNA binding protein 1, a.k.a. EWS) gene.

Translocations involving the chromosomal region 22q12.2 are found in 90-95% of patients with Ewing sarcoma or peripheral primitive neuroectodermal tumors (PNET). Ewing sarcoma is the second most common, highly malignant bone tumor in children and young adults. The most frequent translocation involving the EWSR1 gene region is t(11;22)(q24.3;q12.2) juxtaposing the EWSR1 gene in 22q12.2 next to the FLI-1 (friend leukemia virus integration 1) locus in 11q24.3. FLI-1 is a member of the ETS family of transcription factors. Less frequently, EWSR1 can also be fused to ERG, a transcription factor closely related to FLI-1 but located in 21a22.2.

For prognosis and appropriate treatment it is important to differentiate Ewing sarcoma/PNET from classic neuroblastoma, Wilms tumor, and rhabdomyosarcoma. In combination with the histopathological diagnosis, detection of the EWSR1 rearrangements by FISH can be used to confirm the diagnosis of Ewing sarcoma/ PNET.


References
Bridge RS, et al. (2006) Mod Pathol 19: 1-8.
Delattre O, et al. (1992) Nature 359: 162-5.
Lee J, et al. (2005) Cancer Genet Cytogenet 159: 177-80.
Rekhi B, et al. (2012) Virchows Arch 461: 687-97.
Romeo S & Dei Tos AP (2010) Virchows Arch 456: 219-34.
Sandberg AA & Bridge JA (2000) Cancer Genet Cytogenet 123: 1-26.
Yang L, et al. (2012) Hum Pathol 43: 1463-70.
Zucman J, et al. (1993) EMBO J 12: 4481-7.

## **Probe Description**

The SPEC EWSR1 Dual Color Break Apart Probe is a mixture of two direct labeled probes hybridizing to the 22q12.1-q12.2 band. The orange fluorochrome direct labeled probe hybridizes proximal and extends inward into intron 4 of the EWSR1 gene, the green fluorochrome direct labeled probe hybridizes distal to that gene.




Ideogram of chromosome 22 indicating the hybridization locations.

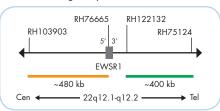


SPEC EWSR1 Probe map (not to scale).

### **Results**

In an interphase nucleus lacking a translocation involving the 22q12.1-q12.2 band two orange/green fusion signals are expected representing two normal (non-rearranged) 22q12.1-q12.2 loci. A signal pattern consisting of one orange/ green fusion signal, one orange signal, and a separate green signal indicates one normal 22q12.1-q12.2 locus and one 22q12.1-q12.2 locus affected by a 22q12.1-q12.2 translocation.

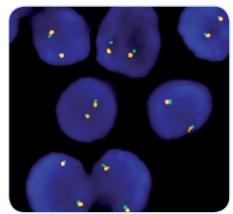

SPEC EWSR1 Dual Color Break Apart Probe hybridized to normal interphase cells as indicated by two orange/green fusion signals per nucleus.

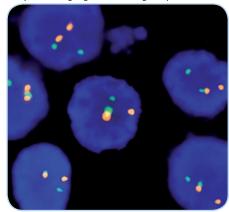

Ewing sarcoma tissue section with translocation affecting the 22q12.1-q12.2 locus as indicated by one non-rearranged orange/green fusion signal, one orange signal, and one separate green signal.

Prod. No.	Product	Label	Tests* (Volume)
Z-2096-50	Zyto <i>Light</i> SPEC EWSR1 Dual Color Break Apart Probe C€ IVD	•/•	5 (50 µl)
Z-2096-200	Zyto <i>Light</i> SPEC EWSR1 Dual Color Break Apart Probe C€	•/•	20 (200 µl)
Related Products			
Z-2028-5	Zyto Light FISH-Tissue Implementation Kit C E IVD Incl. Heat Pretreatment Solution Citric, 150 ml; Pepsin Solution, 1 ml; Wash Buffer SSC, 210 ml; 25x Wash Buffer A, 50 ml; DAPI/DuraTect-Solution, 0.2 ml		5
Z-2028-20	Zyto Light FISH-Tissue Implementation Kit C    IVD  Incl. Heat Pretreatment Solution Citric, 500 ml; Pepsin Solution, 4 ml; Wash Buffer SSC, 560 ml; 25x Wash Buffer A, 100 ml; DAPI/DuraTect-Solution, 0.8 ml		20

<sup>\*</sup> Using 10 µl probe solution per test. C E IVD only available in certain countries. All other countries research use only! Please contact your local dealer for more information.